



Equipment Protocol for Extended Shutdown

Operatories

1. Empty water bottle and put the water bottle back on the delivery unit to purge all water from handpieces and A/W syringes until air only comes out. Turn the dental unit off.
2. Take water bottle off and let sit upside down to air dry.
3. Purge all scalers/prophy jets and turn off.
4. Turn off X-rays – unplug Pans/Conebeams.
5. Change all solids collectors and run an enzymatic cleaner through vacuum lines.

Sterilization

1. Empty water thoroughly from sterilizers/statims – refer to manufacturer's instructions.
2. Shut off water to distillers and water purifiers. Follow the manufacturer's instructions for extended periods of non-use.
3. Drain and clean Ultrasonic cleaners – unplug.
4. Wipeout and clean handpiece cleaners – unplug.
5. Hydrim – remove soap/salt - run shipping cycle - we can provide detailed instructions.

Mechanical Rooms/N2O closet

1. Turn off all Nitrous and Oxygen tanks/turn off the manifold alarm.
2. Shutoff water to wet-ring vacuum pumps and dry vac tank cleanouts (make sure water lines are turned back on before used again). Refer to manufacturer's instructions for non-use.
3. Turn off air compressor.

Lab

1. Cad/Cam mills – Follow manufacturer's instructions for non-use.
2. Run water through model trimmer to clean out trimmer and lines.
3. The plaster trap will begin to smell without use.

Midway will provide support for your office. If you have any questions on any of the above maintenance, **call 1.800.474.6111 Option#2**

If you do not have specific manufacturer instructions for your equipment, we can send you the information. Please provide the Model # and serial#.

We are working on a protocol to open your office.

Text for Service 844.737.8448