



## Equipment Checklist for Office Startup

### Mechanical Rooms/N2O Closet

- Turn on manifold alarm Nitrous and Oxygen tanks. Check gauges. Make sure flowmeters and scavenging are off in operatories. Check gauges in tank room to ensure it stays pressurized.
- Change solids traps on your vacuum before turning on.
- Turn on water valves to vacuum pumps and turn on. Test vacuum in operatories. Check for vacuum leaks. Do this before you turn on air compressor.
- Check amalgam separator to make sure its working properly. The filter may have to be replaced.
- Turn on air compressor – Check gauge on compressor to see if it gets up to pressure.

### Operatories

- Check o-rings on HVE and saliva ejector. Lubricate as necessary.
- Change solids collectors. Run hot water for 20-30 seconds through HVE and saliva ejector in each room.
- Fill water bottles with shock treatment. Remove DentaPure/Sterisil straws and follow the instructions for each straw for shock treatment.
- Run each handpiece and a/w syringe to ensure they have water and no leaks.
- Check chair functions and presets.

### Sterilization

- Refill sterilizers/statims with distilled water and run cleaning cycle per manufacturer's instructions.
- Turn on water to distillers and water purifiers. Follow the manufacturer's instructions to shock.
- Hydrim - Reconnect soap if it was disconnected. Run a cycle and check for leaks.
- Fill ultrasonic cleaner and run foil test.

### Lab

- Cad/Cam mills - Contact us to get protocol from manufacturers.
- Run water through model trimmer to make sure there are no leaks.

Midway will provide support for your office.

If you have any questions on any of the above maintenance, call 1.800.474.6111 option #2

We can set up a facetime call for quick questions to avoid unnecessary service calls.

If you do not have specific manufacturer instructions for your equipment, we can send you the information.

Please provide the model # and serial #.

**Text for Service: 844.737.8448**